


bebird

Enable everyone to enjoy visible ear cleaning


Start bebird T15

iOS Search for "bebird" in App Store
Android Search for "bebird" in Google Play




1 Scan QR code to download the APP

2 Open the cover
Power on automatically




3 Open the APP
Connect device WiFi




Connect WiFi

1 Connect product WiFi
"bebird-T15-xxxxx"




2 Back to bebird APP
Click on "GO" to use it

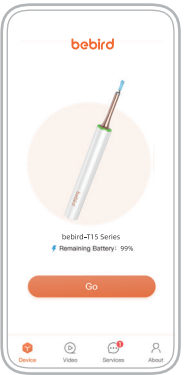

Replace the ear spoon head

Rotate the ear spoon head, pull out and replace the ear spoon and fasten the ear spoon head

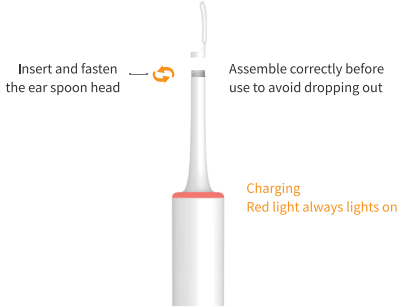


Insert and fasten the ear spoon head

Assemble correctly before use to avoid dropping out


Charging
Red light always lights on

Accessory


1 Metal & Silicone wrapped Ear Spoon (Small) x 1
Harmless ear cleaning




5 Metal pimple squeezer rod x 1
Easily squeeze out acne




2 Double-color Q-elastic Ear Spoon (Small) x 1
Thorough removing of cerumen




6 Sponge Ear Spoon x 2
Removing of earwax and grease in ears




3 Double-color Annular Ear Spoon x 1
Easy Scraping: higher curvature




7 Storage Bin x 1
Store accessories inside



4 Double-color Q-elastic Ear Spoon (Big) x 1
Easy and thorough removing of cerumen



8 Charging Cable x 1
TYPE-C Charging cable




Troubleshooting

Problem	Solution
The product does not work after power-on	Please charge for 85 minutes with the standard USB cable
The camera indicator is not lit	Power off the product and restart it
Crash of APP	Uninstall the APP and scan the QR code to install it again
The accessory holder cannot be installed smoothly	Rotate the ear spoon head, pull out and replace the ear spoon and fasten the ear spoon head
The power indicator is lit in red	Please charge for 85 minutes with the standard USB cable
After successful connection, the APP does not display normally	Disconnect WiFi between the mobile phone and the product, restart the product and connect WiFi again
The WiFi connection is unstable. Sometimes, it disconnects just several minutes after connection	Restart the product and connect the APP again

If the problem persists, please contact our customer service department

Product parameters

Brand	bebird
Product Name	Smart visual ear-cleaning Rod
Model	bebird T15
Rod weight	25g
Network standard	IEEE 802.11b/g/n
Antenna	Internal FPC antenna
Working frequency	2.4GHz
Image transmission rate	20fps
Image sensor	CMOS
Working temperature	-10 ~ 45°C
Battery	300mAh lithium battery
Battery life	85 minutes
Charging time	85 minutes
Input current	5V --- 0.5A
Lens diameter	3.5mm
Optimum focal length	10 ~ 20mm
Depth of field	10 ~ 50mm
Pixel	5 Mega-pixels
Gyroscope	6 axis



Attention

- When cleaning lens, please wipe it carefully with professional alcohol cotton swab.
- Make sure the ear spoon is well installed before using, to prevent the ear spoon from falling off.
- Please don't use it while you are moving, and protect from being hit by others.
- It is forbidden for children to use this product alone in order to avoid accidental harm.
- Please do not put the product in liquid to avoid damage.
- Product built-in polymer battery, so it must be charged once a month If it is not used frequently.
- Avoid product exposure to sunshine directly, especially the accessories, in order to avoid softening.
- During using it, the temperature of the product will rise slightly(Data from bebird lab, the maximum temperature of the machine doesn't exceed about 35°C), please rest assured to use.

Warranty

User info

User name: _____ Telephone number: _____

Address: _____

Seller: _____ Telephone number: _____

bebird

Enable everyone to enjoy visible ear cleaning

⚠ Note: Do not use the charging equipment with output voltage exceeding 5V

Hazardous substance

Part name	Hazardous material					
	(Pb)	(Hg)	(Cd)	(Cr(VI))	(PBDE)	(PBDE)
Printed Circuit Components	X	○	○	○	○	○
Case	○	○	○	○	○	○
Plastic rubber accessory	○	○	○	○	○	○

This form is formulated in accordance with SJ/T 11364
 ○: represent the hazardous material ratio is below GB/T26572-2011
 X: represent the hazardous material ratio is over GB/T26572-2011