



XLIFE
Suitable for all family

1
Download the APP

iOS: Search "XLIFE" in App Store
Android: Search "XLIFE" in Google Play





OR scan the QR code to download the app



2 Open APP
Click Xlife x1

3 Press and hold the power button to turn on the ear cleaner






Blue light flashes

Connect to product WiFi


1 Connect Product Wi-Fi "Xlife-X1-XXXXXX"

2 Back to Bebird app Enter and start to use it

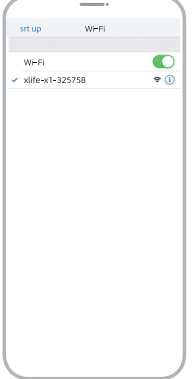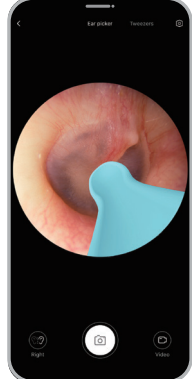Accessory replacement

Unscrew the ear spoon head and pull out the silicone ear spoon for replacement.
Screw and plug the ear spoon head well.




Please check that the ear tip is tight before using it.

The ear tip has its own thread and can be unscrewed clockwise for replacement.


For a better user experience, we recommend replacing the ear tip at least every 3 months. (This can be adjusted according to the frequency of use.)

Accessory


1 PC Ear Tip x 2

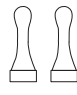


2 Sponge Ear tip x 2
Removing of earwax and grease in ears

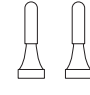


3 Annular Ear Tip x 1
Easy Scraping; higher curvature




4 Silicone Ear Tip x 6



5 Storage Tube x 1
Store accessories inside



6 Charging Cable x 1
TYPE-C Charging cable

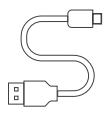

Troubleshooting

Problem	Solution
No response after startup	Please use the incidental charging cable to charge.
The app exits quickly	Delete the existing app and scan QR to reinstall it
Feel tricky with replacing ear tips	Align the replacement ear tip to camera and tighten clockwise
Indicator light flashes	Please use the incidental charging cable to charge.
Abnormal display in app	Interrupt the connection, restart the product to connect again

If you still have problems with the above procedures, please contact our Customer Service Representatives directly for further assistance


Hazardous substance

Part name	Hazardous material					
	(Pb)	(Hg)	(Cd)	(Cr(VI))	(PBB)	(PBDE)
Printed Circuit Components	×	○	○	○	○	○
Case	○	○	○	○	○	○
Plastic rubber accessory	○	○	○	○	○	○

This form is formulated in accordance with SJ/T 11364
○ : represent the hazardous material ratio is below GB/T26572-2011
× : represent the hazardous material ratio is over GB/T26572-2011

Product parameters

Brand	Xlife
Product Name	Smart visual ear cleaner
Model	X1+
Lens Level	High Precision Endoscope
Depth of field	10 ~ 50mm
Lens diameter	3.5mm
Image transmission rate	20fps
Battery life	45 minutes
Charging time	45 minutes
Input current	5V --- 0.5A
Gyroscope	6-axis
Network standard	IEEE 802.11b/g
Working temperature	-10 ~ 45°C
Rod weight	13g
Working frequency	2.4GHz
Image sensor	CMOS



△ Attention

- Please use cotton swab to wipe the lens carefully when cleaning.
- The ear tip cover must be installed properly, not loose before use to prevent it from falling off.
- Please observe the surrounding environment before use. Do not use it in an environment that is susceptible to impact.
- Prohibit children from using this product alone to avoid inadvertent injury.
- Please do not put the device inside any form of liquid.
- The battery must be charged once a month for effective use.
- The temperature will rise slightly when using.
- The product temperature will rise slightly when using (from bebird laboratory data, the maximum temperature of the whole machine does not exceed about 45 °C). Please feel free to use.


Warranty

User info

User name: _____ Telephone number: _____

Address: _____

Seller: _____ Telephone number: _____



△ Note: Do not use the charging equipment with output voltage exceeding 5V